

# TROCARS

## SAFE HANDLING INSTRUCTIONS


### ONE

The trocar shall be used and handled only by a trained physician




### TWO

Ensure that the incision is large enough to accommodate the trocar.

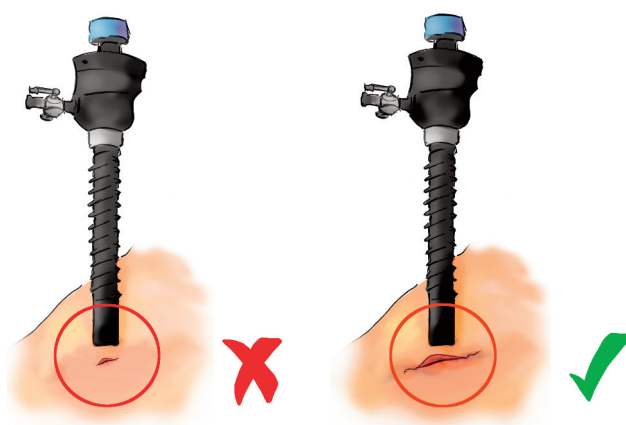

### THREE

Do not grip the trocar very strongly and avoid placing heavy finger pressure or heavy objects on the cannula or any other parts of the instrument.

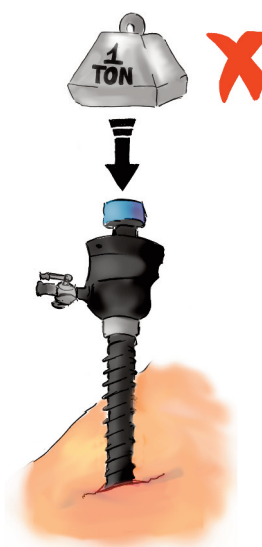


# TROCARS

## SAFE HANDLING INSTRUCTIONS


### FOUR

Always insert and remove the trocar and also any instruments which are to be used with the trocar slowly and cautiously and at the correct angle - perpendicular

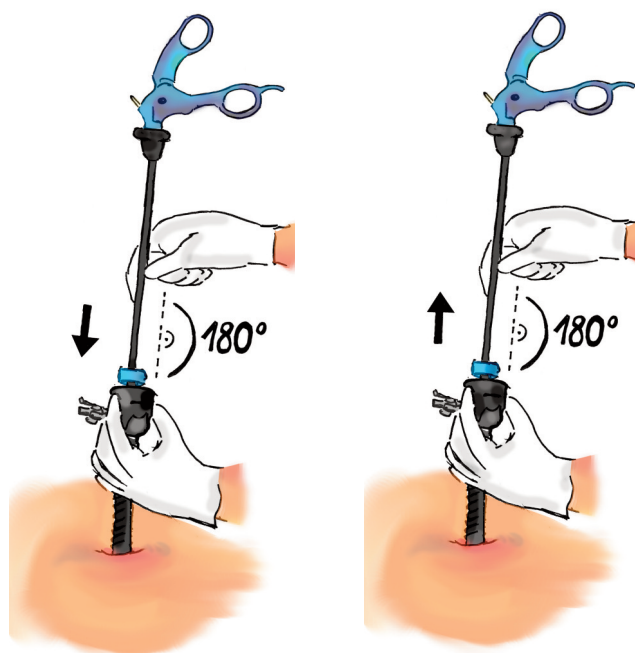


### FIVE

In case of unusual resistance during insertion or removal, stop pushing or pulling the trocar to prevent damage.

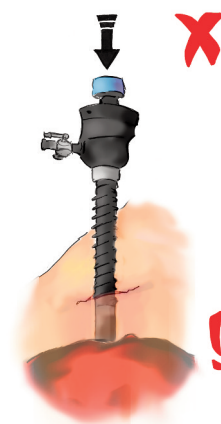

### SIX

Do not bend or twist the trocar and also any instruments which are to be used with the trocar during insertion or removal

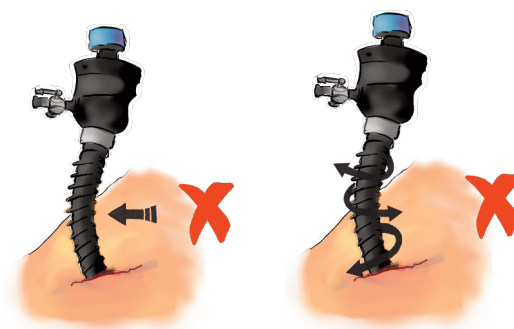


# TROCARS

## SAFE HANDLING INSTRUCTIONS


### SEVEN

Only use instruments which fit through the trocar. Do not force and never try to push larger instruments through the trocar.

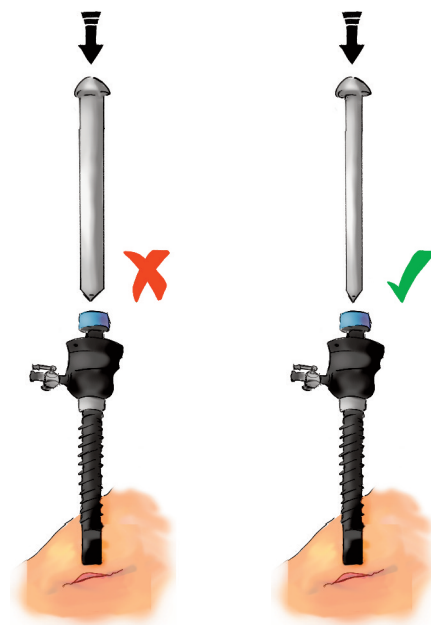

### EIGHT

To use smaller instruments with a larger trocar, always use a reducer.

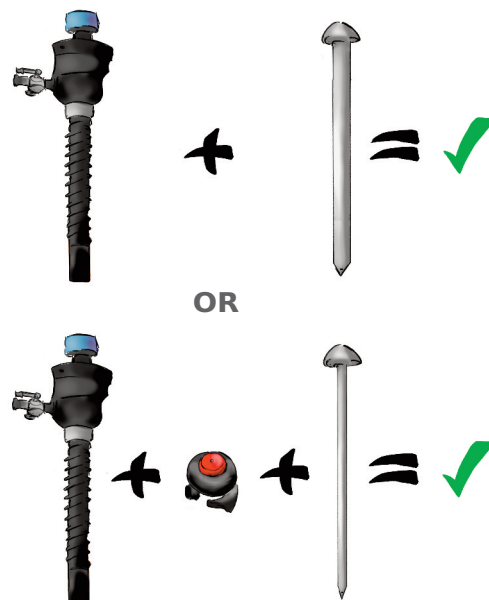

# TROCARS

## SAFE HANDLING INSTRUCTIONS

### NINE

Never use any instruments which are half way inserted into the cannula.

